

## Step 4: Cleaning & Charging

13



Before returning Ring Cutter to the case, clean **Finger Guard** groove with the provided cleaning tool.

14

Wipe the device clean with 70% isopropyl alcohol or 0.5% accelerated hydrogen peroxide wipes.

15



After cleaning, plug-in Ring Cutter and charge for next use.

## Troubleshooting & Indicators



- **DO NOT** force the cutting disc into the ring. Allow the Ring Cutter to cut **automatically**.
- Avoid excessive motion during cutting.
- **DO NOT** use **Open/Close Buttons** during cutting.



10  
Seconds

**Amber lights indicate improper use.** If amber lights flash, pause cutting and resolve the issue. The Ring Cutter will stop cutting for 10 seconds if improper use continues. Cutting can resume after **amber lights** have cleared.

- Examples of improper use:
- Insufficient lubrication
  - Moving or maneuvering the Ring Cutter during cutting
  - Forcing the cutting disc into the ring



The **Status Light** displays the state of the Ring Cutter:

- » Ring Cutter is ready for use
- » Improper use (see above)
- » Apply lubrication
- » Critical battery (plug-in)

When the battery level becomes critically low, the device will disable the **Trigger**, but will still allow the **Open/Close Buttons** to operate. Plug the device in to immediately enable the **Trigger**.



Scan to order more AssureCUT™ Discs and Ring Lubricant



Scan to register your Ring Rescue Kit



Scan here for Training Videos

RingRescue

Any questions? Contact us at:  
[info@ringrescue.com](mailto:info@ringrescue.com)  
[ringrescue.com/help](https://ringrescue.com/help)



RDD01-P077-V10


## Quick Start Guide



RingRescue | Dolphin®  
Ring Cutter



# Product Overview

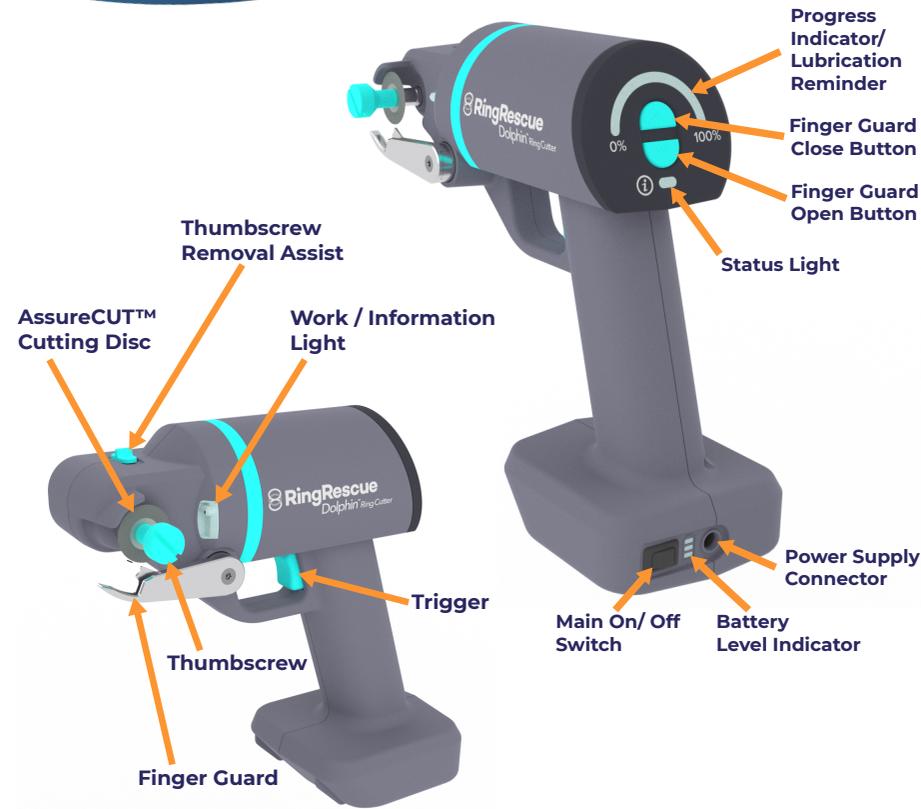

## Training Video



Scan the QR code or visit [ringrescue.com/tutorial](http://ringrescue.com/tutorial) to watch the training video.

## Step 1: Setup & Startup

**Attention!** Regularly monitor ring temperature during cut. If ring is warm, add lubricant or pause cutting. Use only between 10 - 25°C (50 - 77°F) ambient temperature. **DO NOT** touch cutting disc during operation.

- 1** Turn On. If battery level is low, the Ring Cutter can be operated plugged-in. **DO NOT** touch cutting disc during operation.
- 2** Hold Open Button to open Finger Guard. Use a new AssureCUT™ Disc for each patient. To install a new disc: I. Remove Thumbscrew II. Install disc III. Reattach Thumbscrew firmly (finger tight).
- 3** Progress Indicator/Lubrication Reminder. Finger Guard Close Button. Finger Guard Open Button. Status Light. Main On/Off Switch. Battery Level Indicator. Power Supply Connector. Trigger.

## Step 2: Ring Cutting Operation

- 4** Position the patient's hand and the Ring Cutter on a flat, stable surface. A typical ring takes 3 - 6 minutes to cut. The Ring Cutter can cut rings up to US size 13.5 - For tips cutting uncommon rings, see Technical Reference.
- 5** Insert Finger Guard under the ring until ring touches the rear of the guard. Hold the Close Button to close the Finger Guard until the disc touches the ring.
- 6** Apply 2 sprays of Ring Lubricant directly to the cutting disc before beginning the cut.
- 7** Pull Trigger to begin cutting. The Ring Cutter will cut the ring automatically! Monitor your progress with the Progress Indicator.
- 8** The Lubrication Reminder will periodically flash blue during cutting. Apply 2 sprays of Ring Lubricant to the disc when indicated. You can continue cutting while applying the Ring Lubricant.

## Step 3: Completing A Cut

- 9** Expect 3 - 6 minutes cutting time for standard size rings.
- 10** The Progress Indicator will flash green when cut is complete. Hold the Open Button to remove Finger Guard from under the ring.
- 11** Most rings require two cuts for removal. Rotate ring 180° and repeat steps 5 - 10.
- 12** Once the ring is removed, discard the used disc. AssureCUT™ Discs are single patient use only. Sharps Disposal.

**2** AssureCUT™ Discs are SINGLE USE ONLY

Instructions continue on back side